




How to Set & Change Ventilator & Alarm Settings on VOCSN Ventilators

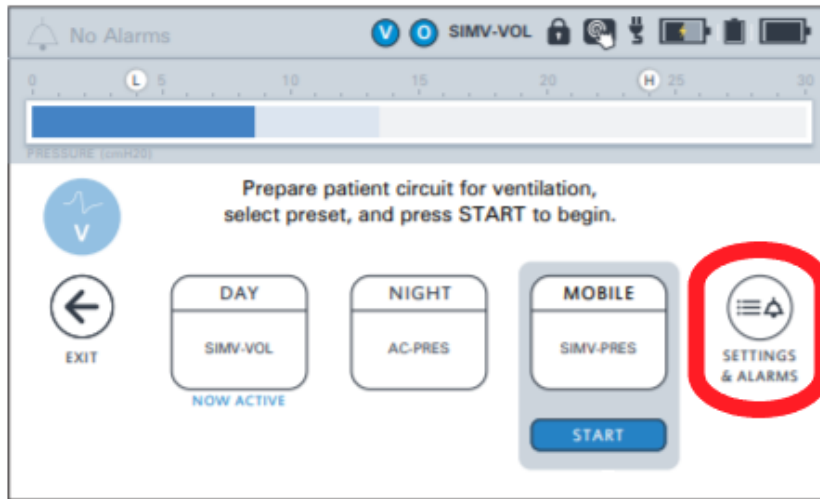

Press here if you need to change a control

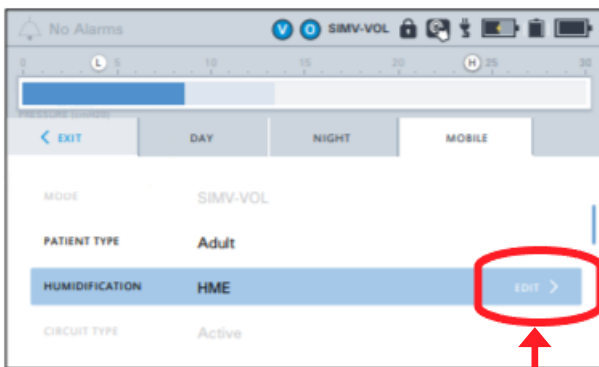

Select EDIT to scroll to find the Mode, Rate, Tidal Volume, PEEP and FiO2 control that needs to be modified

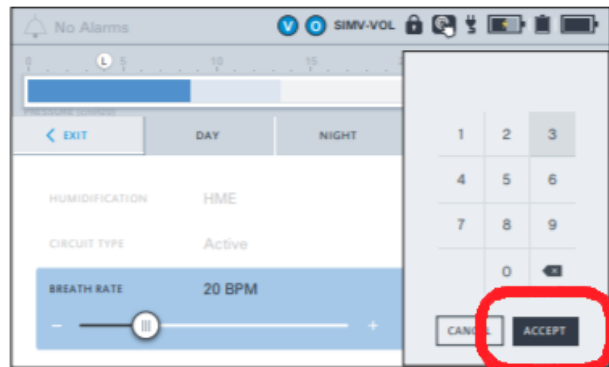


Select ACCEPT to confirm the changes

Select START to activate the changes



Note: Follow the same procedure to modify settings and alarms.

How to Set Oxygen Controls on VOCSN Ventilators

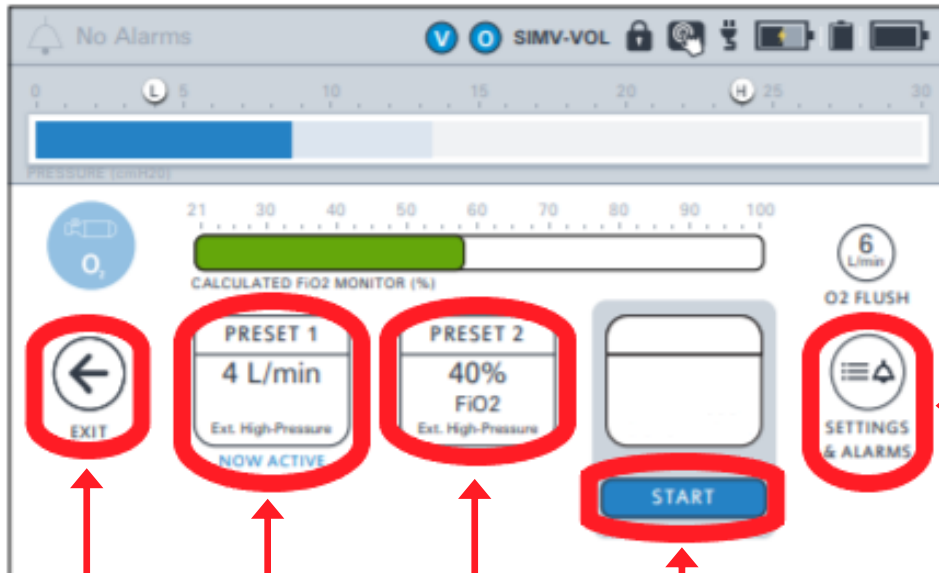

1
Press here if you need to change a control

2
Select to set or modify L/min being delivered

3
Select to set or modify FIO2 being delivered

4
Select START to activate settings

5
Press here to exit this screen