


# How to Change Key Settings on Draeger Infinity

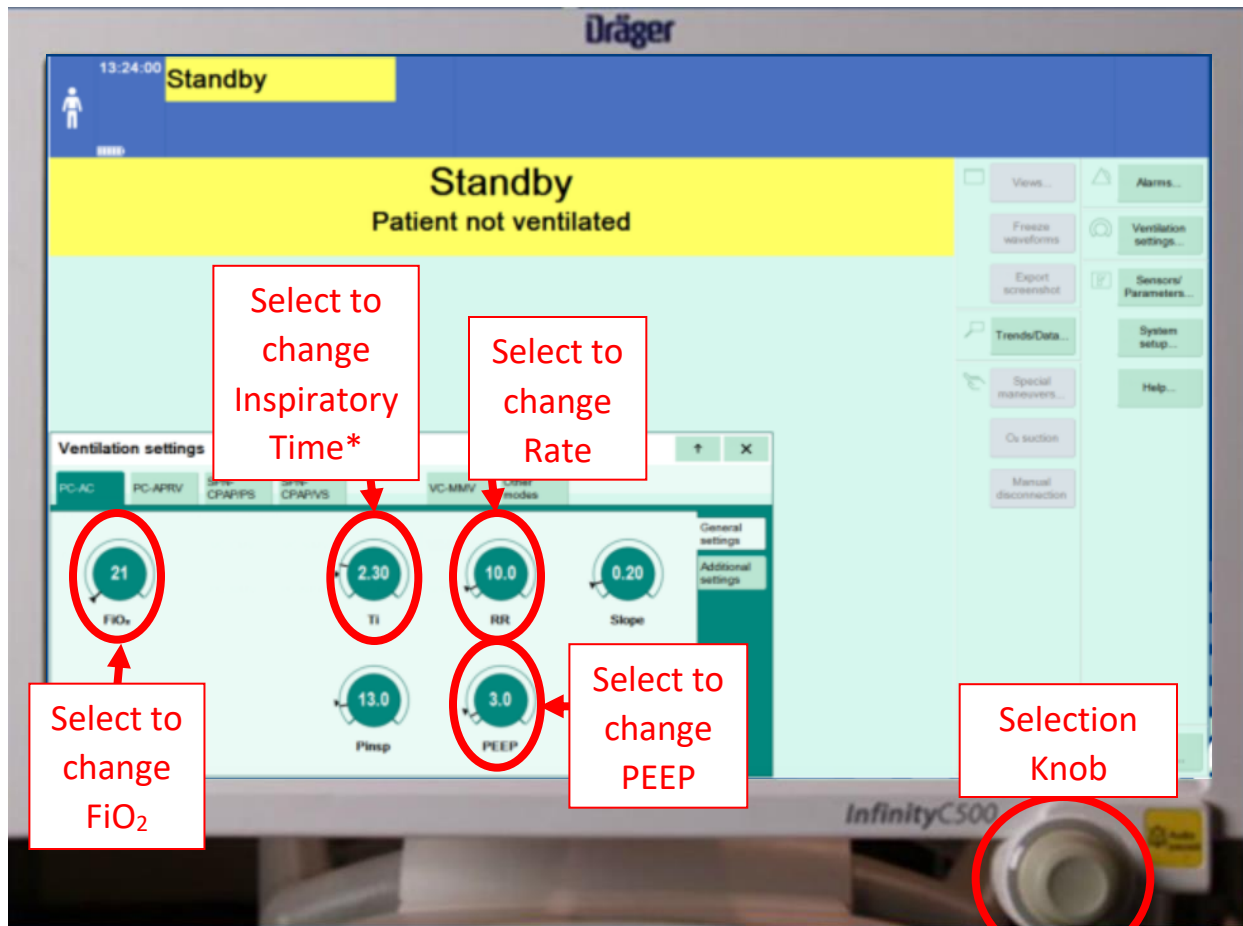

\* change Inspiratory Time to change calculated Tidal Volume

## Setting ventilation parameters

- Press the desired therapy control. It will turn yellow.
- Set the value by turning the rotary dial knob and press to confirm.

## Additional Settings for ventilation

The ventilation modes may be enhanced with additional settings to optimize ventilation.

## Selecting Additional Settings

- Press the Additional settings tab. The additional settings of the active ventilation mode are displayed.
- Press the tab of the relevant additional setting page. The page for setting the corresponding parameters is displayed.
- Use the tab or buttons to activate or deactivate the additional setting.
- Press the corresponding therapy control button.