


How to Change Key Settings on Servo-i Ventilator

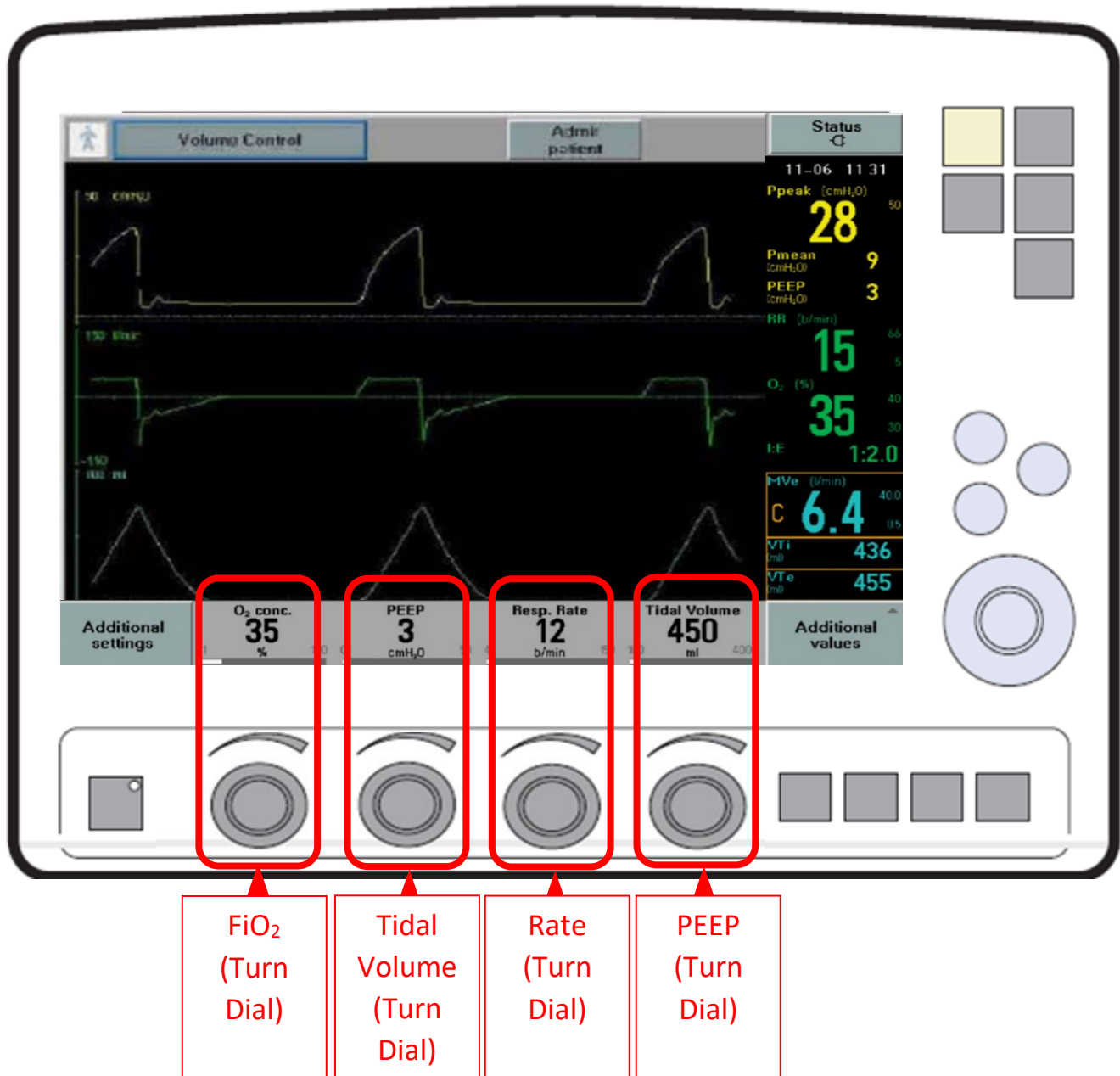


To adjust a breathing parameter directly:

1. Turn the Direct Access Knob corresponding to the parameter you wish to change until the desired value is displayed on the screen.

The Main Rotary Dial and Direct Access Knobs become inoperative for 2 seconds when the user reaches a defined safety limit for the parameter being adjusted.

WARNING! When you adjust a breathing parameter using a Direct Access Knob, the parameter will change immediately starting with the next breath; no additional confirmation is required.

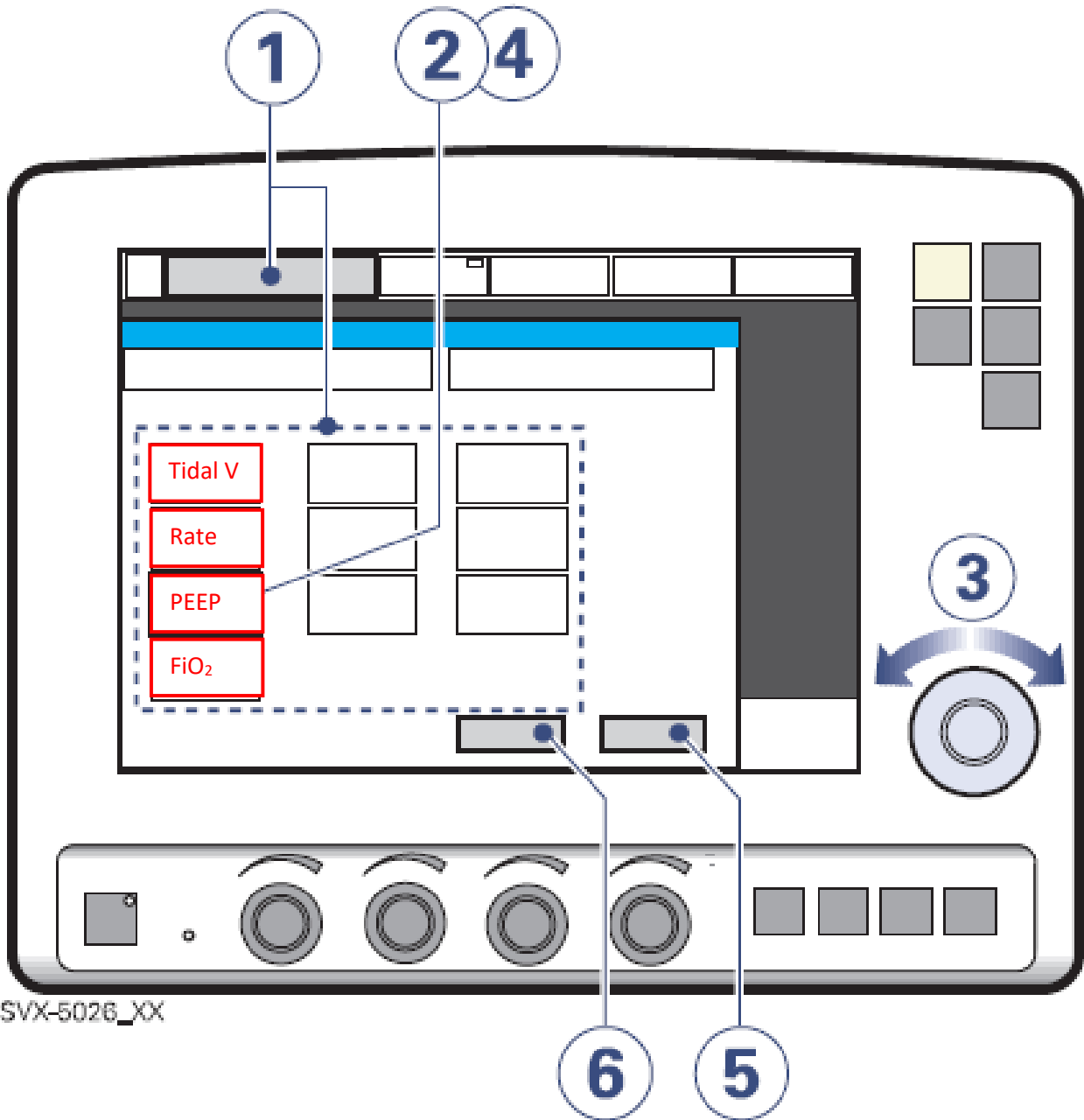

SVX-5026_XX

To adjust ventilator settings:

1. Open the desired window by pressing one of the touchpads on the screen or one of the fixed keys.
2. Activate the desired touchpad by pressing it.
The touchpad is now highlighted in white with a blue frame and it is possible to set a new value.
3. Turn the Main Rotary Dial to the desired value.
4. Confirm the settings by pressing the parameter touchpad or by pressing the Main Rotary Dial.
5. The touchpad turns grey again indicating that the new setting has been entered.
6. Press *Accept* to activate the new settings, or *Cancel* to start over.